

DEAF ALERTER[®] NIGHT CRADLE and Bedroom Kit

User Instructions



Deaf Alerter Night Cradle

The **DEAF ALERTER[®]** Night Cradle wakes you up when you are asleep in bed. It is connected to a pad under your pillow at night. It will vibrate when there is an alarm.

It has a battery, so it will still work if there is an electricity power cut.

How to use your Night Cradle

1. Plug the pillow pad into the back of the **Night Cradle** where marked
2. Plug the power unit into the back of the **Night Cradle** where marked and into the mains socket.
3. Switch on. The red light comes on meaning it is correctly attached.
4. Insert the **Deaf Alerter** into the top of the box, with the belt clip facing down and the display facing out.
5. The green light will come on when **Deaf Alerter** is plugged in correctly.
6. When there is a fire alarm, the pillow pad will vibrate and wake you up.
7. Press the button on the top of the **Deaf Alerter** to see the message.

If you want to see a BSL version of these instructions please go to the Accommodation webpages.